DR-HO'S

neck pain *pro*

User manual | P. 2

Manuel d'instructions | P. 14

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Bedienungsanleitung | P. 20

Thank you

Thank you and congratulations on the purchase of your new DR-HO'S Neck Pain Pro! You have made a smart and important decision with regard to self-treatment and care. Many people have been pleasantly surprised with the degree of improvement they have experienced with this state-of-the-art device, and you too will discover how it helps to improve your quality of life and reduce tension.

DR-HO'S Neck Pain Pro is an elaborate but simple-to-use device. From chronic to acute muscle tension and aches, this unique device provides quick and effective relief from many of your conditions.

Our aim is to develop unique and effective products that help people improve their quality of life. I would like to take this opportunity to express our sincere thanks to you, our customer, for your support.

DR-HO'S Neck Pain Pro is a medical device class IIA so please read all guidelines carefully and keep it for future reference.

If you have any questions or require assistance with your DR-HO'S Neck Pain Pro, please contact us via info@drhonow.eu or  .

Yours in health & happiness,

Dr Michael Ho
Pain & Headache Specialist

TABLE OF CONTENTS

Safety precautions.....	3	Cleaning & maintenance	11
Information on intended use	8	Problem-solving.....	12
Specifications	8	Disposal.....	14
Treatment programs.....	9	Symbols.....	14
Batteries.....	10	Electromagnetic compatibility.....	15
Electrode pad guidelines	11	Warranty & customer service	16

SAFETY PRECAUTIONS

Warnings

The long-term effects of electrical stimulation are unknown.

PLEASE CONSULT YOUR DOCTOR BEFORE USING THIS DEVICE. THE DEVICE MAY BE DANGEROUS IF USED BY CERTAIN SUSCEPTIBLE INDIVIDUALS.

DO NOT USE THIS DEVICE

- If you have a cardiac pacemaker, implanted defibrillator, or other implanted metallic or electronic device. Using the device could then cause electric shock, burns, electrical interference, or death.
- Near magnetic resonance imaging (MRI) equipment. The device is MR Unsafe.
- Together with medical electronic devices such as an artificial heart or lung or respirator.

- In the presence of electronic monitoring equipment (e.g., cardiac monitors, ECG alarms), because it may cause them to malfunction.
- On open wounds, rashes, swollen, red, infected or inflamed areas or skin eruptions (e.g., phlebitis, thrombophlebitis, varicose veins) or on top of or close to cancerous lesions.
- Over insensitive areas of skin.
- On the opposite sides of your head, because the effects of brain stimulation are unknown.
- By applying the pads to your head, face, the front and sides of your neck, your throat, or on both sides of your thorax at the same time.

SAFETY PRECAUTIONS

- To stimulate the carotid sinus nerves, particularly in patients with a known sensitivity to the carotid sinus reflex.
- To stimulate your front neck or mouth, because this could cause severe muscle spasms resulting in closure of your airway, difficulty in breathing, or adverse effects on heart rhythm or blood pressure.
- Across your chest because the effect of electrical current on the heart may cause cardiac arrhythmias.

DO NOT USE ON THESE INDIVIDUALS

- Children or infants, because the device has not been evaluated for pediatric use.
- People who have lost their mental capacity.
- Pregnant women.

DO NOT USE THIS DEVICE DURING THESE ACTIVITIES

- When in the bath or shower.
- While sleeping.
- While driving, operating machinery, or any activity which might cause involuntary muscle contractions that may put the user at undue risk of injury.

WARNINGS AND PRECAUTIONS REGARDING THE PADS

- Apply pads to normal, healthy, dry, clean skin only. Do not apply pads to wounds or damaged skin because that may disrupt the healing process.
- If you experience any skin irritation or redness after using this device, do not continue to stimulate in that area of the skin until it returns to normal.
- Both of the pads in a set must be placed on the body at the same time in order to work. Make sure the pads are not touching each other because otherwise no impulses will be felt.

PRECAUTIONS

- If you feel discomfort or abnormal symptoms during treatment, stop using the device and consult a doctor before resuming treatment.
- If you wish to move the electrodes to another location of your body during a treatment cycle, turn the device off first, then move the electrodes and resume treatment.
- If you have any problems setting up, maintaining or using this device, contact customer

SAFETY PRECAUTIONS

- service and report anything unusual. Do not try to repair the device yourself.
- Keep the device out of the reach of infants, children or pets. Any inhalation or swallowing of small parts is dangerous and can be potentially fatal.
- Only use accessories that have been specified or authorized by the manufacturer. The use of unauthorized accessories may cause damage to the device or danger to the user.
- Long cables have the potential to strangle. Be careful when using cables around the neck and shoulder areas.
- Do not bend or fold the pads, as this may cause the pads to malfunction.
- Always store the pads on the transparent protective film when not in use.
- Do not apply any solvent or any other liquids (except water) to the pads or to your skin because these will cause the pads not to stick and to malfunction. The pads are pre-gelled and will adhere to your skin when moist.
- Do not use the device in areas of high humidity, such as the bathroom or in the shower, as this may damage it.
- Do not attach the electrode to the same area of the skin for longer than 40 minutes each time, as this may cause skin irritation.
- During treatment ensure that metallic items with conductive capabilities, such as necklaces, belts and rings, are kept away from the electrodes.
- Make sure the wires are properly plugged into the device and connected and that the pads are stuck tightly onto the skin area you wish to treat to ensure effective therapy.
- If using rechargeable 1.5 V batteries, make sure you use a CE approved battery charger. Never connect the device directly to a battery charger or to other powered equipment.
- The simultaneous connection of a patient to high frequency surgical equipment may result in burns where Pain Therapy Device electrodes have been attached and possibly damage the Pain Therapy Device.

SAFETY PRECAUTIONS

- There should be no contact between pads.
- Do not leave pads attached to the skin after treatment.
- Do not use the device in the presence of a flammable anesthetic gas mixture, compressed oxygen or nitrous oxide.
- If the device is not functioning properly, or if you feel discomfort during use, you should stop using it.
- Only use the device for its intended purpose.
- Do not use the device while wearing other electronic devices, as this may damage it.
- Do not use the device near a mobile phone, as this may cause it to malfunction.
- Do not bend or pull the end of the cord.
- Replace the cord when broken or damaged.
- Do not throw the batteries into a fire because they may explode.
- Dispose of the device, batteries and components in accordance with applicable legal regulations. Unlawful disposal may cause environmental pollution.
- Do not use the stimulator during pregnancy because it is not yet known whether it is safe to do so.
- If you have suspected or diagnosed heart disease, seek advice from your doctor.
- If you have suspected or diagnosed epilepsy, seek advice from your doctor.
- Do not use the device:
 - a. If hemorrhage is likely.
 - b. Following recent surgical procedures if stimulation may disrupt the healing process.
 - c. Near your eyes, to your head or face.
- Electrode placement and stimulation settings should be based on the instructions of the prescribing practitioner.
- You should only use the leads and electrodes recommended for use by the manufacturer.

ADVERSE REACTIONS

You may experience:

- Skin irritation or hypersensitivity due to the electrical stimulation or electrical conductive gel.

SAFETY PRECAUTIONS

- Skin irritation and burns underneath the electrodes when applied to the skin.
- 1. You should stop using the device and consult your doctor if you experience any adverse reactions.
- 2. Do not treat one area for extended periods of time (more 40 minutes a session, up to 3 times/day) because your shoulder muscles may become exhausted and sore.
- Infants and children.
- People who have lost their mental capacity.

HOWEVER, IF YOU FEEL YOU WOULD BENEFIT FROM USING DR-HO'S NECK PAIN PRO AND WOULD LIKE TO GIVE IT A TRY, PLEASE CONSULT YOUR DOCTOR.

CONTRAINDICATIONS

The following people should not use the device:

- Patients who have a cardiac pacemaker, implanted defibrillator, or other implanted metallic or electronic (auxiliary) devices. Using the device could then cause electric shock, burns, electrical interference, or death.
- People with abnormally high blood pressure.
- Patients with undiagnosed pain syndromes.
- People with a tendency for internal bleeding.
- People with epilepsy.
- Pregnant women.

INFORMATION ON INTENDED USE

DR-HO'S Neck Pain Pro is a new generation multifunctional device based on physics, modern microelectronics and clinical practices. It stimulates healthy muscles and relieves tension, thereby improving and facilitating muscle performance and temporarily relieving pain. It uses electrical current to stimulate nerves and relieve chronic, intractable pain and pain associated with arthritis relating to tension at the back.

IMPORTANT!

Brief information on intended use: This Pain Therapy Device (Model: P.T.S-IIB) is intended to relieve pain associated with sore or aching muscles in your lower back, shoulders, arms and

back of the neck due to strain from exercise or ADLs (activities of daily living), principally regular household and work activities.

Note: this product is certified as medical device in the European Union under the Medical Device Directive 93/42/EEC by SGS CE1639, exclusively for the indication(s) of relief of pain associated with sore or aching muscles. Other non-medical uses escribed to this device are not within the scope of CE certification, and users should be aware product performance and/or safety has not been evaluated by SGS for those purposes.

SPECIFICATIONS

Power source	DC 3.0V, 2 AAA batteries
Method of line current isolation	Type BF applied part
Number of output channels	1
Number of output programs	3
Output intensity level	5 steps
Timer range	10, 20, 40 min
Weight	Pain Therapy Device: 74g, Neck Pain Pro: 470g, Small body pads: 10g, Foot relieve pads: 100g, FlexTone pads: 37g
Dimensions	Pain Therapy Device: 12 x 7 x 2 cm, Neck Pain Pro: 48 x 27 x 10 cm, Small body pads: 46 x 46 mm, Foot relieve pads: 260 x 110 mm, FlexTone pads: 80 x 120 mm

SPECIFICATIONS

Operating environment	Temperature 5~40°C, Humidity ≤80%RH, Atmospheric pressure 86~106kPa
Storage environment	Temperature: Pain Therapy Device -20 ~ 55°C, Electrode 5~40°C, Humidity ≤95% RH, Atmospheric pressure 50~106 kPa
Shelf-life	Electrode pads: 1 year, Foot relieve pads: 3 years, Pain Therapy Device and Neck Pain Pro: 3 - 10 years (depending on how well it is maintained)

TREATMENT PROGRAMS

Therapies:	A	B	C
	To temporarily increase local blood circulation and relaxation.	Symptomatic relief and management of chronic, intractable pain and of pain associated with arthritis. Stimulation of healthy muscles to improve and facilitate muscle performance.	
How long does the therapy last?	This depends on the timer setting chosen. The therapy ends automatically.		
How do I personalize the therapy?	You select program and intensity yourself.		
How long should I use the device and how often?	Each session should last 30 to 40 minutes. No more than 3 sessions per day.		
When should I stop using it?	When the pain has subsided or stopped, if treatment is uncomfortable, in the event of an adverse reaction, and if the condition worsens.		

Note: many people experience immediate relief while others require regular use to feel the benefits. The results vary and will depend on your underlying conditions and how often

you use the device. If your pain does not subside, you can try increasing the intensity or you can change the program.

BATTERIES

Insert 2 AAA batteries before use:

1. Remove the battery cover on the back.
2. Insert batteries correctly so that the markings correspond.
3. Refit the battery cover.

Important Notes

Low battery power will cause the device to lose its effectiveness. Please change to new batteries whenever stimulation weakens. Alkaline batteries are recommended as a stable power source.

If the indicator lights fade, or if the stimulation feels weak on your skin, this may be an indication of low battery power.

WARNING!

Replace only with the same or equivalent type of batteries, as recommended by the manufacturer:
2 x AAA (1.5 V) alkaline batteries.

Do not dispose of the batteries with normal household waste, but hand them in at an official collection point in accordance with local regulations governing the disposal of such materials.

Remove the batteries to prevent them from running down whenever you are planning to store the device for more than 3 months.

If the device is used once a day for about 30 minutes per treatment, one pair of AAA Alkaline batteries should last 3 to 6 months.

We recommend the use of Alkaline batteries because they provide a steady power level until it runs out. Non-Alkaline and rechargeable batteries have a tendency to decrease in power level over time, and you might mistakenly think the device is not working right because the treatment is not as strong.

Please refer to the Quick Guide for pad placement and user instructions.

ELECTRODE PAD GUIDELINES

- Only use the original electrode pads, gel and wires supplied by the manufacturer because other electrode pads and wires may not be electrically compatible with your device.
- The pads should only be used on one person to avoid skin reactions or cross-contamination.
- Always turn the device off before removing or repositioning the electrodes.
- Wash skin thoroughly and then dry it before applying the electrodes.
- Apply the whole surface of the electrode pads firmly to the skin.
- Do not use electrode pads that do not stick properly to the skin, or only partially stick to the skin.
- If the skin looks red after treatment, you should not treat that area until the redness has disappeared.
- The electrode pads can be reused 30 to 50 times depending on your skin type and other conditions. Whenever you notice that the pads are not sticking properly to your skin, or if the gel pads feel dry, you should dab the gels with a small amount of water by hand or using the spray bottle (do not wash them with water directly). This will help restore the adhesiveness and conductivity of the electrode pads. If moistening the pads fails to make them sticky again, they may need to be replaced.

CLEANING & MAINTENANCE

DR-HO'S Neck Pain Pro and Pain Therapy device:

- Use a wet cloth or neutral cleanser to wipe the surface.
- Do not use volatile liquids such as alcohol, thinner or gasoline.
- Keep well away from moisture, high temperatures, direct sunlight or water droplets.
- Please do not disassemble, repair or rebuild the device yourself because this will invalidate the warranty.
- Remove the batteries from the device if it is not used for more than 3 months.

Body and foot pads:

- Never stick two electrode pads close to each other.
- Keep the electrode pads clean and never expose them to high temperature and direct sunshine.
- Keep the electrode pads at temperatures of 5~40°C.
- Gently clean the electrode pads with a moist cloth or use a brush to remove dust.
- Do not clean the pads with hot water or chemicals.
- Do not touch the gel of the pads with your fingers, as this could reduce their adhesiveness.

PROBLEM SOLVING

Problem	Possible causes	Try this solution
The device does not appear to be working properly.	The two pads are not attached properly.	Attach both pads to the body and make sure the pads are not touching each other.
	Your bare feet are not making proper contact with the foot pads.	Make sure you remove all socks and shoes. Your feet must be bare. Ensure both bare feet are on both foot pads.
	The wires from the electrode pads or the foot pads are not properly connected to the the Pain Therapy Device.	Make sure the wire from the electrodes or foot pads and main wire to the device are connected to each other.
	The wire plug is not firmly connected to the output port at the top of the Pain Therapy Device.	Push the wire plug firmly into the output port of the Pain Therapy Device.
	Weakening electrical pulses.	Turn the ON/OFF intensity dial in a clockwise direction.
	Power is running out.	Replace batteries, even if you bought them recently.
	The electrode pads are not properly in contact with the skin.	Press the electrodes firmly onto the skin.
	The gel surface of the electrode pad is dry or contaminated.	Moisten the gel pads with a little water or replace with new electrode pads.
	Foot pads and bare feet are not sprayed with water.	Spray water onto the soles of your feet and onto the foot pads prior to use.
	Your skin is covered with oil or lotion that is preventing the pads from sticking.	Wash your skin with water and try putting the pads on again.
The device stops working before treatment has ended.	The batteries have run out.	Change the batteries.
	The wire plug has not been fully inserted into the output port.	Reconnect the plug to the output port.
	The wires between the electrode pads or foot pads and the main wire are not connected.	Reconnect the wires from the electrodes or foot pads to the main wire leading to the Pain Therapy Device.

Problem	Possible causes	Try this solution
The skin has turned red and/or itches or prickles.	You are allergic to the electrode pads.	Stop using the machine until you ascertain the cause of the skin irritation.
	You have a small cut or acne.	Do not put electrodes on recently shaved skin, areas that have suffered insect bites, scrapes, acne or any open wounds.
	The adhesive surface of the pads is dirty or too dry.	Wash the adhesive surface of the pads gently with your fingertips for about 3 seconds under slow running water.
	The therapy has gone on too long.	Use the device for a maximum of 40 minutes per treatment and no more than 3 times a day.
	The 2 pads are not attached firmly and fully to the skin.	Attach the 2 pads firmly to the skin to ensure full contact.
	The surface of the pads is worn out.	Replace the set of pads.
Three yellow lights on the Pain Therapy Device are flashing and you cannot feel any stimulation.	One of the pads has become disconnected from your skin or a wire is unplugged.	Reconnect the body pads to your skin or reconnect the wire. Turn the Pain Therapy Device off and then turn it back on again to restart the device.
Three green lights are flashing and you cannot feel any stimulation.	The auto shutdown timer has activated and the device has powered of.	Turn the power dial until it clicks to the OFF position first. Then turn the power dial back to the ON position.
Batteries have been installed but the light does not turn on.	The battery compartment is dirty and the connectors are blocked.	Examine and clean.
	The batteries are not at full power and have not been installed correctly.	Replace the batteries or install them correctly in the compartment.
	There is a problem with the connectors in the battery compartment.	Try pulling the connectors outward slightly using a small pair of pliers.
	The power dial has been turned OFF and then ON.	Turn the power dial OFF and then ON again.

DISPOSAL

1. The Pain Therapy Device, Neck Pain Pro and accessories should not be thrown away with household waste, but should be recycled by you or the supplier.
2. Dispose of packing materials appropriately in accordance with the rules and regulations in your area to prevent harm to the environment.
3. Please stop using the pads if the individual packaging has been open for more than three months, or if the pads lose their adhesiveness.

SYMBOLS

Symbol	Meaning
	SERIAL NUMBER. This symbol will be accompanied by the manufacturer's serial number.
	CONSULT INSTRUCTIONS FOR USE.
	Indication of conformity with the essential health and safety requirements set out in European Directives 0120 and 1639 – Notified body identification (SGS).
	DATE OF MANUFACTURE. This symbol will be accompanied by the date of manufacture.
	MANUFACTURER. This symbol will be accompanied by the name and the address of the manufacturer.
	AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY. This symbol will be accompanied by the name and the address of the authorized representative in the European Community, adjacent to the symbol.
	Dispose of separately from other household.
	Type BF applied part.

Symbol

Meaning

	IP classification.
	ATTENTION! Caution! Follow operating instructions! Failure to do so could place the patient or operator at risk.
	Batch code.
	MR UNSAFE. This device cannot be taken into the controlled access area of an MRI system.

ELECTROMAGNETIC COMPATIBILITY

DR-HO'S Pain Therapy Device has been tested and found to comply with the electromagnetic compatibility (EMC) limits for medical devices. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation.

CAUTION: do not use this device simultaneously with devices that have high EMI levels.

“Reduce pain and become more active. We hope you enjoy this total body pain relief system for many years to come!”

Dr Michael Ho

WARRANTY & CUSTOMER SERVICE

We hereby guarantee that if this product malfunctions or fails in any way within one year from the date of purchase, the purchaser will not be charged for any of the parts needed and labor costs incurred to repair the defective device, if it is used under normal conditions and in accordance with this instruction manual. This warranty is only valid if the purchaser can provide proof of purchase.

Please visit www.drhonow.eu to register your DR-HO'S Neck Pain Pro. You will be asked to enter the Serial Number of your product, along with the purchase date and name of the shop of purchase. The registration will enable us to contact you in the unlikely event of a product safety notification. By registering your product you acknowledge to have read and understood the instructions

for use, and the warnings set forth in the accompanying instructions. To find the serial number, check the backside of the Pain Therapy Device. If you cannot find the serial number, please send an email to customerservice@drhonow for assistance.

Before you send your device back, please contact us about the defective device and send us a description of the problem, plus a copy of your sales receipt (as proof of purchase) via info@drhonow.eu. We will then investigate whether the problem can be resolved by our experienced customer support. In the event that, for some reason, your device cannot be repaired, we will replace it for you at no additional charge during the one-year warranty period.

Imported and distributed by
Dr Ho Europe B.V.
1261 XR - 5, the Netherlands
info@drhonow.eu | www.drhonow.eu

For customer service:
E: info@drhonow.eu

Are you happy with your DR-HO'S Neck Pain Pro®? Share your posts, tips & tricks! Like & follow us @drhonow & tag your posts

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#MyDrHo #DrHoLovers #DrHoNow

Fast relief for headaches, tension, neck & shoulder pain.

WARNING: DR-HO'S equipment is not intended to diagnose, treat or cure any disease or condition. If you experience pain or health issues, always consult your doctor for the right diagnosis and treatment.

PRECAUTIONARY MEASURES: Do NOT use DR-HO'S equipment or other electrostimulators if you wear a pacemaker or are pregnant. If you have any questions, please consult your doctor before using the device.

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Guang Zhou Xinbo Electronic Co Ltd.
MedNet EC-REP GmbH