Stand Up Chair

Rise to comfort and vitality with the Vitadora Stand Up Chair!

VitaDora

LIVE. BETTER. LONGER.

Stand Up Chair

Congratulations!

You have just purchased the Vitadora Stand Up Chair , a product designed with quality, safety and functionality in mind. Please read this manual carefully before using this product, especially the safety chapter, so that you can use it correctly and enjoy your new Vitadora Stand Up Chair to the fullest and safely. Keep this manual for future reference when necessary.

2. Safety instructions

We recommend that you read all instructions carefully before you start using your new Stand Up Chair

- Check the compatibility of the mains and machine voltages before starting to use it.
- The chair operates at a voltage of 12 V, which is completely safe for human use. Make sure that the product is connected to an outlet having the same configuration as the plug.
- Unplug the product when not in use.
- Do not use the chair outdoors.
- Do not carry this appliance by supplied cord as a handle.
- Keep the cord away from heated surfaces.
- Close supervision is necessary when this appliance is used by on or near children or people with disabilities.
- This unit is not a toy. Children should not use it or play with it.
- Do not stand on the chair.
- Do not clean with water or chemicals. When cleaning, do not forget to unplug it first.
- Stop using the chair immediately in case of discomfort.
- If the use of this elevator chair causes discomfort, it should be discontinued immediately without attempting to manipulate the mechanism.

Health caution

- If you have any concerns regarding your health, consult your doctor before using this product.
- If you experience pain in a muscle or joint for a prolonged period of time. Discontinue use and consult your doctor. Persistent pain could be a symptom of a more serious condition.
- For people with a pacemaker: A recliner typically contains electric motors and electromagnetic fields, which should not directly interfere with the functioning of a pacemaker, as modern pacemakers are well shielded against such influences. However, it is always wise to consult your doctor to ensure there are no

specific risks in your situation, especially if your pacemaker or medical condition is unique.

Attention!

- Only one person can sit in the chair at a time.
- Do not use this chair if you notice that the cables, screws or anchors are in poor condition. Regularly check the condition of cables, screws and anchors to ensure the optimal functioning of your chair.
- Do not use the chair if any part is missing or broken.
- This chair should only be handled or repaired by an authorized technician.
- The box with the transformer and remote controls for your chair is located on the underside of the chair, between the chair's frame.

3. Before using your chair recliner for the first time


1. Thoroughly clean the removable parts with a damp cloth. Never use abrasive products to clean the chair.
2. Place the chair on a flat floor. When moving the chair, consider the type of flooring. For floors prone to scratches, it is advisable to attach felt pads under the legs of the chair to prevent damage. We do not accept responsibility for damage to the floor caused by the chair.
3. Never lift your chair by the armrests. When moving the chair, lift it from the bottom of the armrests or the side panel.
4. Ensure that the main power supply has sufficient power for the voltage required by the chair.
5. Avoid direct sunlight or placing your chair near a fireplace/wood stove to prevent discoloration or distortion of the chair.

4. Assembly of the chair.


- To facilitate handling, the Stand Up Chair is delivered in several parts that can be easily assembled.
- Unpack all parts. Place them on the floor in an unobstructed location, preferably in the area where you intend to permanently place it.
- Assembly is very simple. We recommend having two people do this.

5. Steps for chair assembly

1. Carefully slide the backrest piece of the chair until it fits snugly into the base.
2. Connect the wires as shown in the figure (connectors).
3. Do not force any connection as each plug only fits its mate (the one of the same color)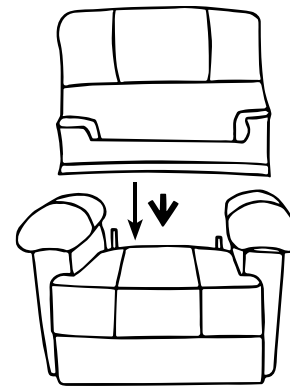

Assembly

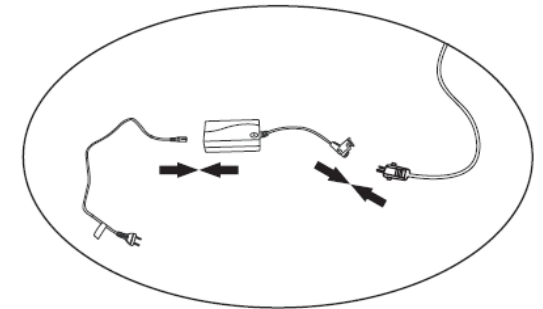Connectors

6. How to recline your chair, fold down the footrest and elevate the chair

1. Both the backrest recline, footrest folding and elevation functions are automatic.
2. You will only need to press the buttons on the controller that controls these functions (Figure A).
3. Using the button  you can recline the backrest at the same time as the footrest is lowered. As it is automatic, you only must release the button when you reach the desired position.
4. To return to the initial position, press the button . Holding the button down will start to raise the chair until the person occupying the chair is almost standing.
5. Starting from the initial position, if you want to use the elevating function, press the button  until you reach the desired elevating position.
6. To return to the initial position, press the button .

7. Remote control

How to use

1. Insert adapter jack into the socket found on back of the chair, then plug the adapter into a wall outlet
2. Turn the unit ON by pressing the POWER button once, The LED will illuminate when power is on.

8. Maintenance instructions

To store:

Place the unit in its box or safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric. Do not wrap the power cord around the unit. Do not hang the unit by the power cord.

To clean:

- Ensure the unit is unplugged from the power source, and allow to cool before cleaning.
- Do not place in any liquid to clean. Clean fabric cover by wiping with a damp sponge, using a mild detergent or suds. Pat dry with clean, dry cloth.
- Do not use abrasive cleaners, brushes, gasoline, kerosene, glass / furniture polish, paint thinner or too hot water to clean the unit.
- To clean the unit, make sure the massager is unplugged. Allow the unit to cool and the wipe down with a damp clean cloth. Do not use any strong chemical cleaners.

Technical specifications

Dimensions:

Width 85.5cm

Depth 96-155.5cm

Height 102.5-79

Height when lift up 137.5cm

Material: Technology Fabric + Foam

INSTRUCTIONS – AC/DC Switching Power Supply

INTENDED USE

The Switching Power Supply (SPS) are intended to supply home applications or IT equipment and office equipment or medical device with safe extra low voltage. The SPS is an appliance inlet type (or directly plugged in a socket outlet) while the device to be supplied is connected with a cord and plug from the SPS. Observe and follow the instructions and cautions as follows. Otherwise SPS damage and/or serious injury may occur. Save these instructions for further reference. Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

SAFETY INSTRUCTIONS

Connections:

- Only connect with supply voltage equal to the voltage shown on the SPS;
- Only connect with a load which does not exceed the rated load of the SPS;
- Only use the SPS in conjunction with recommended products;
- Duty cycle: Max 10%, Max T_{ON}: 2minutes.
- The Power unit is intended to be correctly orientated in vertical or floor mount position.

Ambient Conditions:

- Operating temperature: 0°C ~ 30°C
- Operating relative humidity: 20% ~ 95%
- Storage temperature: 0°C ~ 60°C
- Storage relative humidity: 20% ~ 95%
- Do not operate near fire or in high temperature ambient;
- Do not use this SPS in a dusty ambient;
- Only operate the SPS in dry rooms indoors.
- Do not expose to water or other conductive liquid electric shock hazard.

Cautions:

- Do not place the SPS near easily inflammable material when it is operating;
- Allow air ventilation around the SPS for cooling. Do not cover the SPS while it is operating otherwise
- overheating will occur;
- Place the SPS out of reach for children. Do not allow children to play with the SPS.

SUGGESTION:

ZB-H290020_A: DC29V 2A

MAINTENANCE:

- Check the power supply cord and output cord for damages before use;
- Check the enclosure for cracks or other damages before use;
- ATTENTION: Do not operate the SPS if the connection cords or the enclosure are damaged; A damaged SPS can not be repaired. Disable the damaged SPS in an obvious way .So that it won't represent a hazard to others. Please dispose off a damaged SPS according to the local regulations;
- The SPS may be cleaned with a damp towel, but may never get in direct contact with water. After
- cleaning, please make sure that the SPS is dried completely;
- Do not use any organic solvents (petrol, alcohol, etc.) for cleaning the SPS. The case of the SPS may
- be damaged and explosive vapour may develop. Do not use any sharp edged objects for cleaning.
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the appliance.

- If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.
- Fuse only replaceable by qualified professional. Symbol explain:

SPECIFICATIONS:

Input voltage range: AC100-240V, 50/60Hz;
Rated Output Voltage: ZB-H290020_A: DC29V 2A;

Symbol explain:

	Double or reinforced insulated protection
	It is Compliance with WEEE directive
	Refer to user's manual before use
	Refer to user's manual before use

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